



Reading an On-Site Drug Test: Negative, Positive & Invalid Results

NEGATIVE

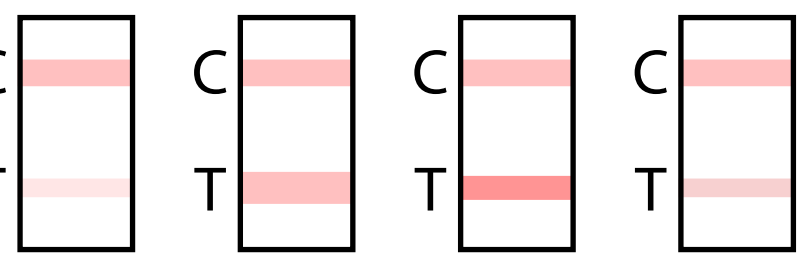


Variations in the thickness or darkness of the test line "T" are meaningless. A control line "C" plus any presence of a test line is a negative drug test result.

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POSITIVE




A control line "C" with *no* test line "T" is a positive drug test result.

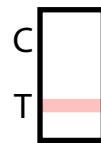

PLEASE NOTE: If the control line "C" is missing the test is considered invalid. If this is the case, please re-test the sample with a new test kit.

THC/Marijuana Testing: Initial vs. Confirmatory Cutoff Levels

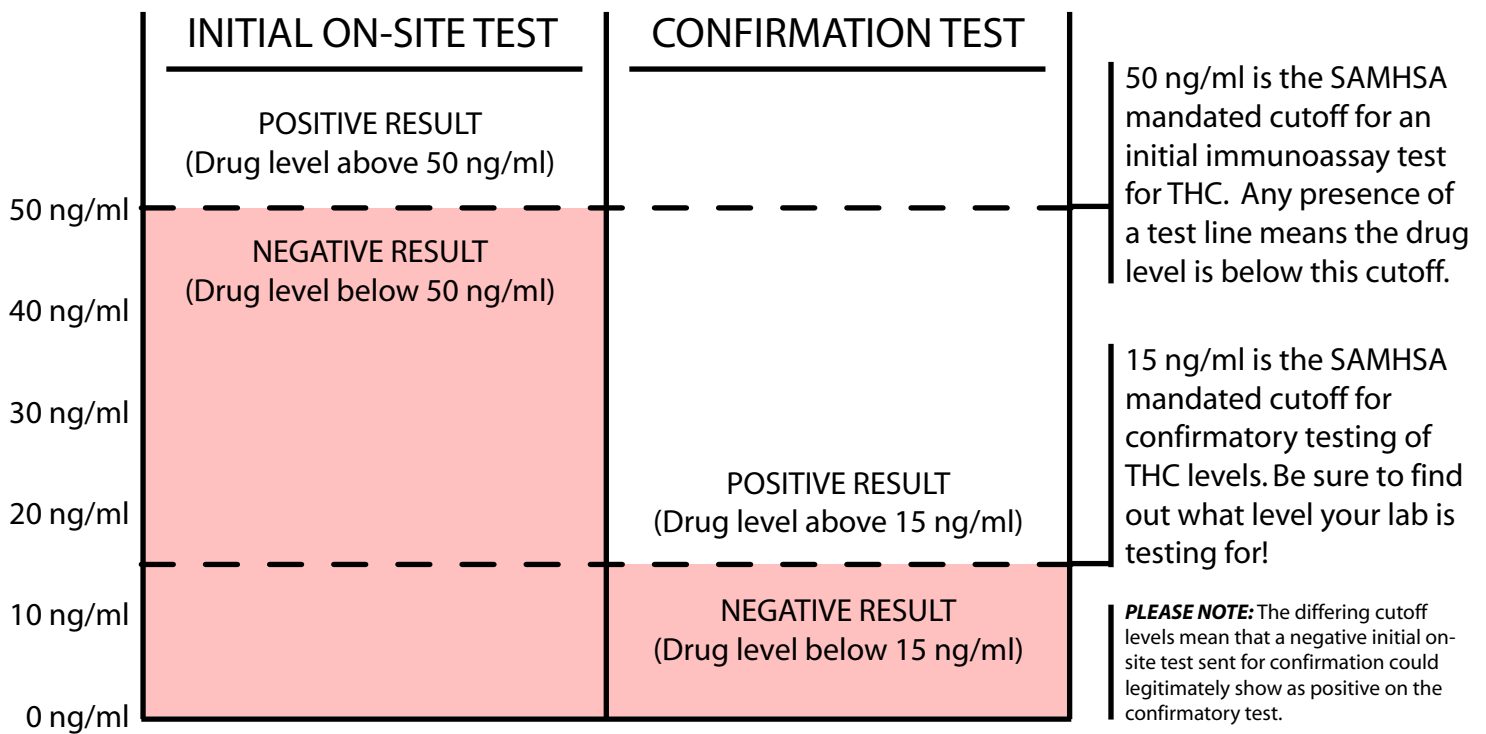

© Drug Detection Devices, Ltd.

(888) 768-6786 toll-free • (770) 886-6226 voice • (770) 886-7792 fax • <http://www.3dl.net>